Operationsliege Surgical Bed

ES Ecoline

Gebrauchsanweisung
User Manual

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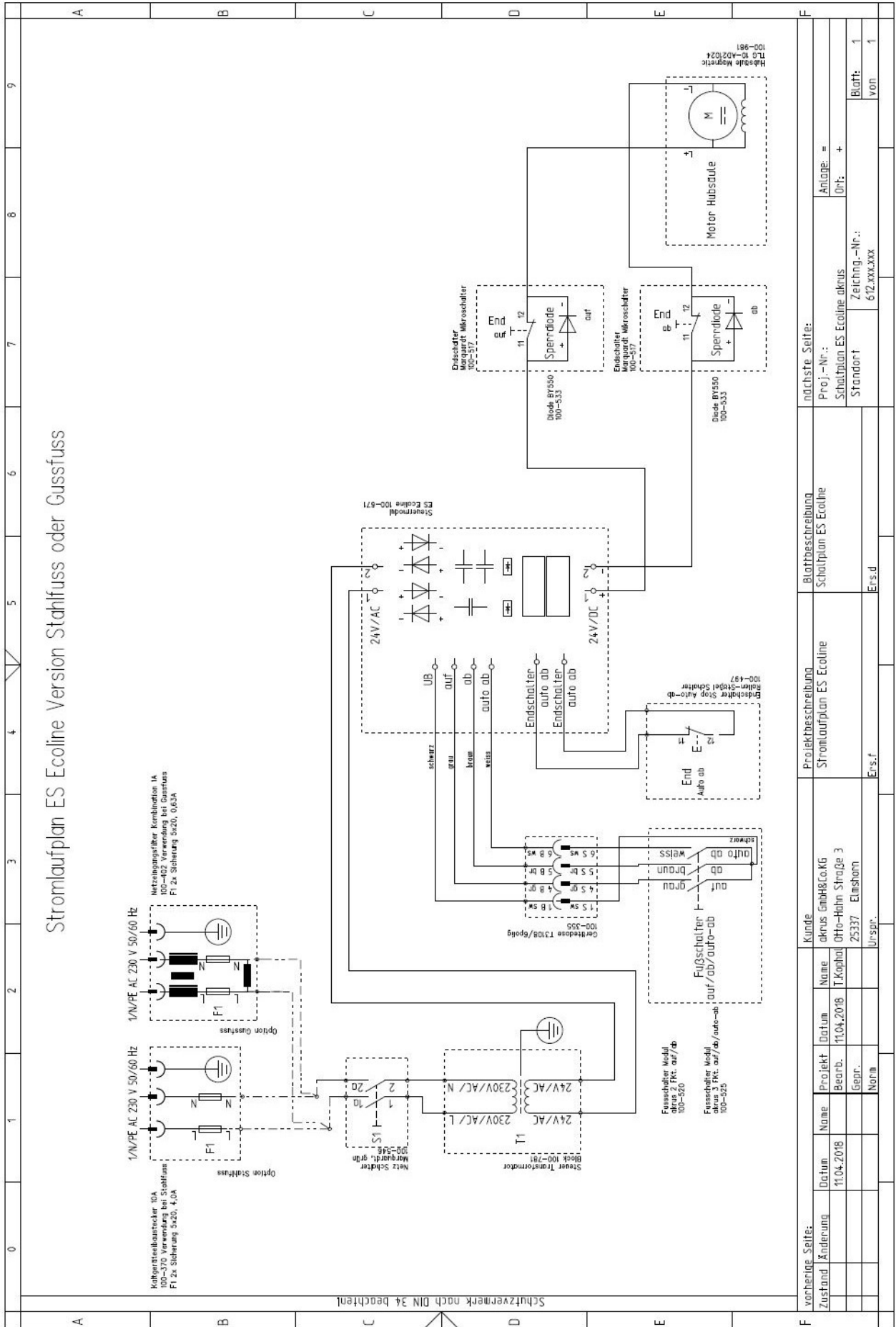
ES Ecoline

User Manual

1. GENERAL	3
1.1. COPYRIGHT©	3
1.2. DISCLAIMER.....	3
1.3. APPLICABLE RULES AND REGULATIONS	3
1.4. WARNING AND CAUTIONING SIGNS	3
1.5. USER INSTRUCTIONS FOR SAFE OPERATION	3
2. SCOPE OF DELIVERY	5
3. INTENDED USE	5
4. SET UP AND INSTALLATION	5
5. OPERATION OF SURGICAL BED	5
5.1. MAIN SWITCH.....	5
5.1.1. DUTY CYCLE (CONTINUOUS OPERATION) OF ELECTRIC ACTUATORS	5
5.2. FOOT SWITCH (STD.) (100-520).....	6
5.2.1. FOOT SWITCH AUTO DOWN BUTTON (100-525)	6
5.3. INGRESS AND EGRESS.....	6
5.4. HEAD REST (241.030690).....	6
5.5. CASTORS (612.030300).....	6
6. ACCESSORIES	7
6.1. OPTIONAL SIDE RAILS FOR ACCESSORIES (612.050016).....	7
6.2. IV POLE (240.030501)	7
6.3. WRIST REST (240.030400).....	7
7. MAINTENANCE	7
8. DISPOSAL OF DEVICE.....	7
9. CLEANING AND CONTAMINATION PROTECTION	8
10. TECHNICAL DATA.....	8
11. MANUFACTURER.....	8
12. CE CONFORMITY.....	8
13. WIRING DIAGRAM.....	8

1. General

To safely operate and control the ES Ecoline please carefully read this user manual before operating. Please pay attention to all cautionary statements pertaining to the safe operation of the ES Ecoline. Please keep the manual on file at all times while the product is in service.

The manufacturer reserves the right to make technical changes or modifications in accordance with advanced technical developments. This manual is not subject to an updating service.

1.1. Copyright©

Distributing, copying or any commercial use of this manual is strictly prohibited unless expressly permitted in writing by the manufacturer. The manufacturer is entitled to compensation for any violation of this right.

The manufacturer claims any and all rights in case a patent is granted or the design is registered.

1.2. Disclaimer

Manufacturer excludes any and all liabilities in case of improper or not authorized operation or maintenance or servicing of the product.

1.3. Applicable rules and regulations

The device complies with the following Regulations for machine products.

- 2006/42/EG
- RL 2011/65/EU

The device meets the requirements of 2006/42/EG and the corresponding national requirements of the machinery directive.

User must observe the national legal requirements governing the prevention of accidents.

1.4. Warning and cautioning signs

This sign indicates a hazard to the equipment!

This sign indicates a hazard to the patient, equipment, or user!

1.5. User Instructions for safe operation

Before using this medical device, please carefully read and observe the safety instructions and recommendations listed in this manual

- To safely use and operate the medical device, use only as directed.
- Operation of the medical device only in accordance with the intended use.
- Do not operate the medical device in explosive or hazardous environments. The medical device must not be operated, with the exception of very minute quantities, in the presence of combustible anesthetics, or volatile solvents like alcohol, benzene or the like.
- Do not set up the medical device in humid or damp rooms. Never expose the product to sprinkling, dripping or splashing water.
- Modifications, authorized by manufacturer, may only be performed by staff authorized by manufacturer. Any modification non authorized or instructed by manufacturer may cause malfunctions or hazards to persons.
- Maintenance or repair work on this product may only be performed by staff authorized by manufacturer.
- Pay special attention to any emphasized safety instruction or information in this user manual.
- Only trained and authorized staff may operate the medical device.
- Only trained service staff authorized by manufacturer may open the medical device.
- No water or other liquids may enter the electric connector housing during maintenance or cleaning procedures. Please refer to section “Cleaning and routine care”.

2. Scope of delivery

The scope of delivery comprises

Position	Qty.
ES Ecoline surgical bed	1
Foot switch	1
Electrical cord	1
Quality test report	1
User manual	1
Options as per order	

3. Intended use

The medical device is intended for use as a surgical bed. Any other use of the product is not permitted.

- The medical device is designed for a maximum patient weight of 135 kg (297 lbs) (EN 60601-2-46 particular requirements for safety of operating tables). A 200 kg (440 lbs) version is available upon request.
- The headrest is designed for a maximum load of 20 kg (44 lbs).
- The chair is suitable to be employed in the immediate proximity of patients.

4. Set up and installation

- Place bed in its designated position. Make sure, floor is level and even. Use adjustable studs to level the bed.
- Check for possible pinching hazards with up and down movements.
- Connect foot switch and secure coupling with safety nut.
- Finally, connect power cord.

5. Operation of surgical bed

5.1. Main switch

The main switch is located next to the power cord socket. After switching on a green light indicates the product ready for operation.

5.1.1. Duty cycle (continuous operation) of electric actuators

The electrical motors are designed for a duty cycle of approximately 6 minutes at full load. Continuous operations exceeding that time period may cause overheating conditions and permanent damage to the transformer.

5.2. Foot switch (std.) (100-520)

The up-down movement is controlled by a foot switch.

5.2.1. Foot switch Auto down button (100-525)

After pressing the auto down (homing) button, the bed will move down to its full stop position. Pressing any button during movement will stop the movement at any time.

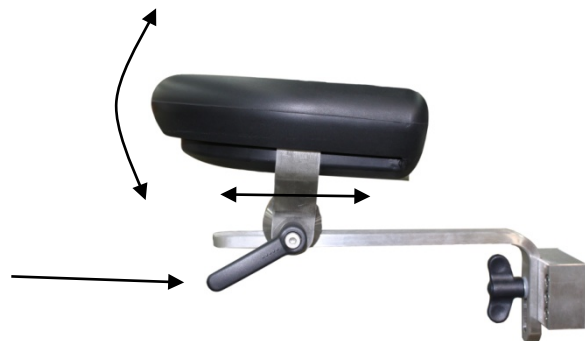
5.3. Ingress and egress

The 60 cm low level position is a very comfortable height to get on and off the pad, even to small handicapped or elderly patients.

5.4. Head rest (241.030690)

The standard head rest features an exchangeable, visco elastic pad.

One central lock bolt controls both, the length extension as well as the self leveling tilt.

5.5. Castors (612.030300)

As an option, 4 pivoting lockable castors are available.

Make sure to pull the power cord before moving the bed out of position,.

6. Accessories

6.1. *Optional side rails for accessories (612.050016)*

Optional stainless steel side rails to attach certain accessories are available. Make sure to secure blocks properly when attaching any equipment to the side rails.

6.2. *IV pole (240.030501)*

6.3. *Wrist rest (240.030400)*

7. Maintenance

The chair is maintenance-free to users and operators.

Product is subject to the regular service intervals. Refer to the current issue of local (national) "Minimum requirements for medical products class I".

8. Disposal of device

The battery and all electrical components must be disposed off in accordance with national applicable laws and regulations.

9. Cleaning and contamination protection

- Contamination protection
In order to protect the product against contamination by liquids, body liquids or other unwanted substances it is recommended to, while in use, cover the product with a non sterile disposable impermeable cover sheet.
- Surfaces are resistant to all commonly used disinfectants as listed in the D G H M (Deutsche Gesellschaft fuer Hygiene und Mikrobiologie) list. www.dghm.org
- Do not attempt to sterilize the SC 5010 SEK
- Clean only the external surfaces of the equipment using a damp cloth. In case of heavy stains do **not** use abrasive or aggressive materials other than common cleansers or detergents. Do not use ether, acetone or concentrated acids.
- No water or cleaning detergents may enter the inside of the equipment.
- To clean soiled upholstery surfaces, use a damp cloth and/or commonly used upholstery spray.
- Before resuming operation, allow sufficient time to dry.

10. Technical data

Voltage	230V/50Hz	<u>Option 120V/60HZ</u>
Actuator:	24V	
Fuses :	6,3	<u>Option 10A</u>
Power input	310 VA	<u>Option 890 VA</u>
Lift:	200 mm	
Lift velocity:	10mm/sec	
Lift capacity, center	2500N	
Weight (depending on accessories)	130 – 180 kg (cast iron base)	
Lowest level :	600mm	
Dimensions (L x W):	1940x620 mm	

Note the dimension are ca. dimension

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Website: www.akrus.de

12. CE conformity

We declare the compliance of the device with the requirements of the Council Directive 2006/42/EG machinery directive

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AKRUS GmbH & Co KG