Patientenstuhl Patientchair

ak 2004 - 5004

ak 5004

ak 2004

ak 3004

ak 4004

Gebrauchsanweisung User Manual

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User Manual

Table of contents

1. GENERAL	3
1.1. COPYRIGHT	3
1.2. DISCLAIMER.....	3
1.3. APPLICABLE RULES AND REGULATIONS.....	3
1.4. WARNING AND CAUTIONING SIGNS	3
1.5. USER INSTRUCTIONS FOR SAFE OPERATION.....	4
1.6. INTENDED USE.....	4
2. PRODUCT DESCRIPTION	5
3. INSTALLATION AND SET UP	5
4. OPERATION	6
5. MAINTENANCE	9
6. CLEANING AND CONTAMINATION PROTECTION	9
7. TECHNICAL DATA	9
8. CE CONFORMITY	9
9. MANUFACTURER	9

1. General

To safely operate and control the patient chair ak 2004 – ak 5004 please carefully read this user manual before operating. Please pay attention to all cautionary statements pertaining to the safe operation of the patient chair ak 2004 – ak 5004. Please keep the manual on file at all times while the product is in service.

The manufacturer reserves the right to make technical changes or modifications in accordance with advanced technical developments. This manual is not subject to an updating service.

1.1. Copyright

© Distributing, copying or any commercial use of this manual is strictly prohibited unless expressly permitted in writing by the manufacturer. The manufacturer is entitled to compensation for any violation of this right.

The manufacturer claims any and all rights in case a patent is granted or the design is registered.

1.2. Disclaimer

Manufacturer excludes any and all liabilities in case of improper or not authorized operation or maintenance or servicing of the product.

1.3. Applicable rules and regulations

The device complies with the following Regulations for machine products.

- 2006/42/EG
- RL 2011/65/ EU

The device meets the requirements of 2006/42/EG and the corresponding national requirements of the machinery directive.

User must observe the national legal requirements governing the prevention of accidents.

1.4. Warning and cautioning signs

This sign indicates a hazard to the equipment!

This sign indicates a hazard to the patient, equipment, or user!

1.5. User instructions for safe operation

Before using this medical device, please carefully read and observe the safety instructions and recommendations listed in this manual

- Before using this medical device, please carefully read and observe the safety instructions and recommendations listed in this manual
- To safely use and operate the medical device, use only as directed.
- Operation of the medical device only in accordance with the intended use.
- Do not operate the medical device in explosive or hazardous environments. The medical device must not be operated, with the exception of very minute quantities, in the presence of combustible anesthetics, or volatile solvents like alcohol, benzene or the like.
- Do not set up the medical device in humid or damp rooms. Never expose the product to sprinkling, dripping or splashing water.
- Modifications, authorized by manufacturer, may only be performed by staff authorized by manufacturer. Any modification non authorized or instructed by manufacturer may cause malfunctions or hazards to persons.
- Maintenance or repair work on this product may only be performed by staff authorized by manufacturer.
- Pay special attention to any emphasized safety instruction or information in this user manual.
- Only trained and authorized staff may operate the medical device.
- Only trained service staff authorized by manufacturer may open the medical device.
- No water or other liquids may enter the electric connector housing during maintenance or cleaning procedures. Please refer to section “Cleaning and routine care”.

1.6. Intended use

The medical device is intended for use as a patient chair. Any other use of the product is not permitted.

2. Product description

The ak 2004- 5004 patient chairs features an infinitely variable height adjustable seat, driven by an encapsulated, maintenance free electric motor and gear box unit. It is suitable for patient examinations. Function control by either a footswitch or directly through electronic control board of the examination equipment.

ak 2004: Backrests and armrests are fixed.

ak 3004: Backrest is fixed. The armrests fold away.

ak 4004: The backrest reclines and the armrests fold away. *The optional footrest is fixed.*

ak 5004: The backrest reclines and the armrests fold away. *The optional footrest is synchronized with backrest movements.*

Option


Power: 110 / 120 Volts 50 / 60 Hz

3. Installation and set up

Attention

If the chair is intended to be connected directly to the examination unit, the connection must be done only by a trained service technician.

- Place chair in it's designated position. Make sure, floor is level and even.
- The auto circuit brake on the connection board must be pressed in.
- Connect foot switch.
- Finally, connect power cord.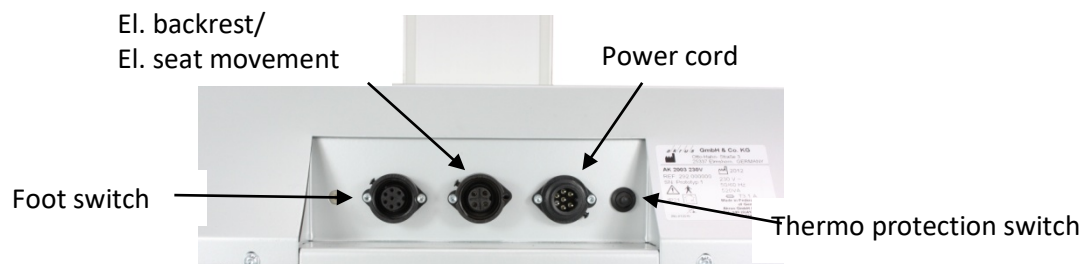


4. Operation


Chair height adjustment controlled by either a footswitch or the control buttons on the examination unit. Foot switch; left button- chair up, right button- chair down.

Option: The middle button is for full stop Down. A short tap on the button moves the chair to the lowest position. To interrupt the Auto-Down-Function tap any button.

Both armrests fold away upwards (ak 3004- ak 5004).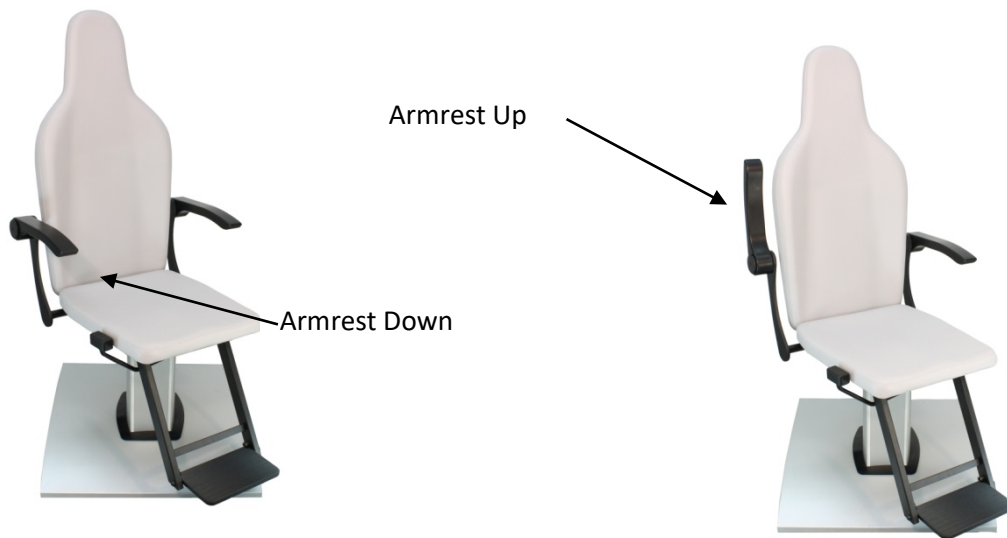

Press the lever behind the backrest sideways to unlock and adjust the backrest infinitely from vertical to fully horizontal (ak 4004 & 5004), or push the switch on each side of the backrest (electrical backrest, only ak 5004).

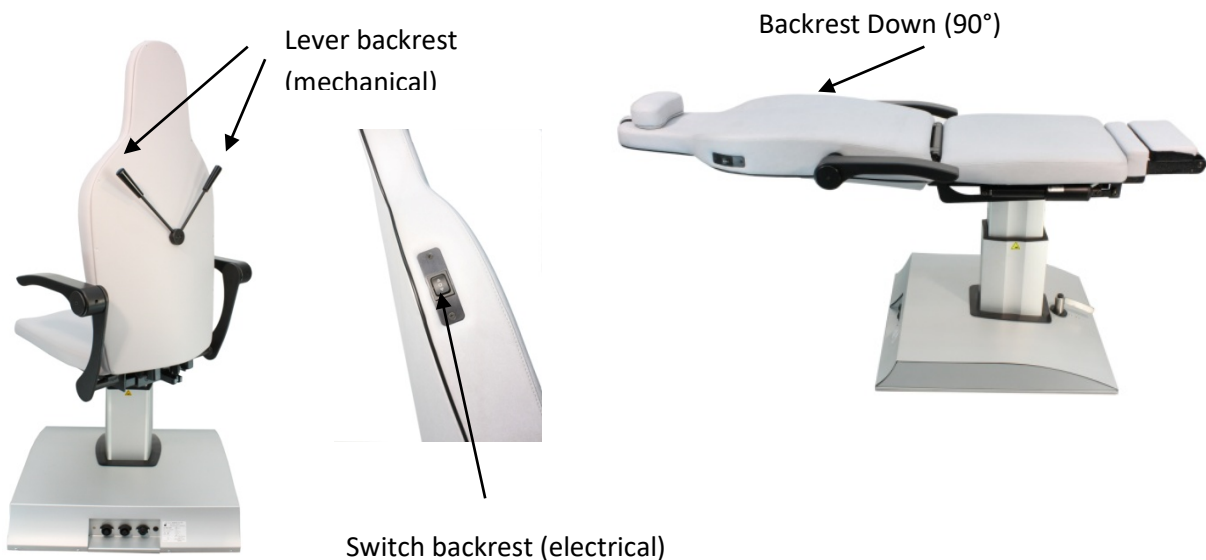
The seat rotates 360° (ak 3004 - ak 5004). It locks every 90°.


Chair 45° (not locked)

The footrest is synchronized with the backrest (only ak 5004). The footrest can be fold away.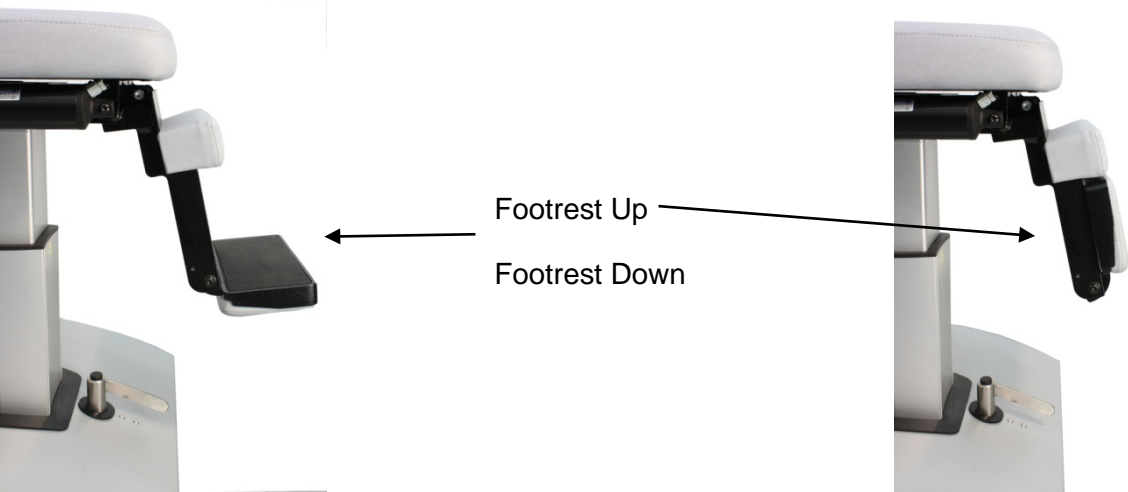


Option: The footrest is attachable (ak 3004 & ak 4004). The footrest can be folded away.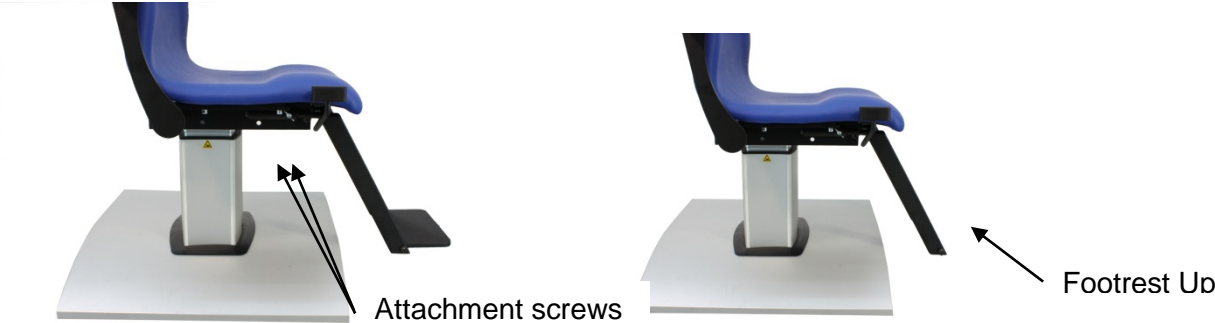

Option: The seat movement unlocks by pressing the lever on the left side (doctors view) (ak 3004 - ak 5004). 3 Positions are available to lock the seat.

The electrical seat movement is controlled by the 5-function-foot switch (ak 3004 - ak 5004).

Option: The moving base plate allows to move the chair whilst unused. (only ak 5004). If the LED shows green, turn the lever left (front view) to unlock the running gear. Move the seat upwards until the LED shows red. Now the Chair can be moved bi-directional. To replace the chair on the floor, follow the steps in reverse order.

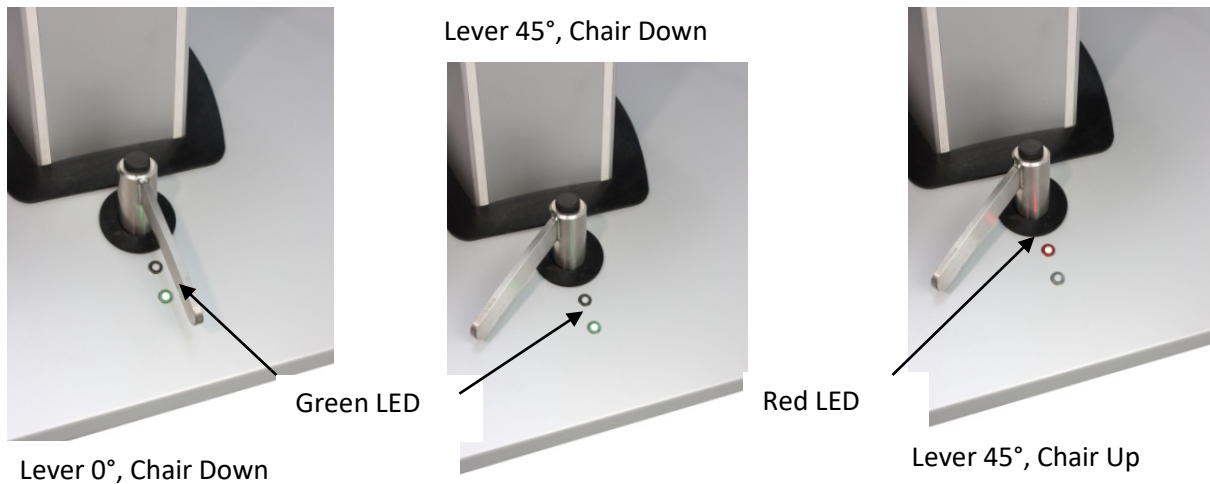Attention

The electric motor is protected by a thermo protection switch. After 2,5 minutes of continued operation, the motor needs a cool down period of approx. 10 minutes. If the thermo protection switch activates, the motor needs a cool down period of approx. 20 minutes (push thermo protection switch afterwards).

5. Maintenance

The product is maintenance free for the end user **however**, bleed current and leak resistance need to be checked every other year or as directed by local laws.

6. Cleaning and contamination protection

- Contamination protection
In order to protect the product against contamination by liquids, body liquids or other unwanted substances it is recommended to, while in use, cover the product with a non sterile disposable impermeable cover sheet.
- Surfaces are resistant to all commonly used disinfectants as listed in the D G H M (Deutsche Gesellschaft für Hygiene und Mikrobiologie) list.
<http://www.gesundheitsamt.de/alle/technik/download/seuche/bakt/list/dghm.pdf>. Any appropriate disinfectant must not exceed the concentrations given for the following ingredients. Propanol=35% // Ethanol=25% This would equal for example Terralin Liquid offered by. Schülke&Mayr.
- Do not attempt to sterilize an ak 2004-ak 5004
- Clean only the external surfaces of the equipment using a damp cloth. In case of heavy stains do **not** use abrasive or aggressive materials other than common cleansers or detergents. Do not use ether, acetone or concentrated acids.
- No water or cleaning detergents may enter the inside of the equipment.
- To clean soiled upholstery surfaces, use a damp cloth and/or commonly used upholstery spray.
- Before resuming operation, allow sufficient time to dry.

7. Technical Data

Electric motor:	220V / 50Hz (120V / 60Hz)
Thermo protection switch	3 A (220V)
Thermo protection switch	3 A (120V)
Rated current	5,5 A
Vertical lift	200 mm
Vertical speed	12 mm / sec
Lifting capacity	150 Kg
Min. seat height	460 mm (ak 2004) – 530 mm (ak 5004)
ttl. dimensions (approx)	(W x H x D):..... 650 x 1360 x 650 mm
KB	1 min/10 min

Product is subject to changes

8. CE Conformity

We declare the compliance of the device with the requirements of the Council Directive 2006/42/EG machinery directive

9. Manufacturer

AKRUS GmbH & Co. KG

Otto-Hahn-Str. 3

25337 ELMSHORN

☎ int. +49 (0) 4121 791930

FAX int. +49 (0) 4121 791939

Email: info@akrus.de

www.akrus.de