Patientenuntersuchungshocker Patient Examination Stool

ak 470 rl, rk, rs

Gebrauchsanweisung
User Manual

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User Manual

Table of contents

1	GENERAL	3
1.1	COPYRIGHT	3
1.2	DISCLAIMER	3
1.3	APPLICABLE RULES AND REGULATIONS	3
1.4	WARNING AND CAUTIONING SIGNS.....	3
1.5	USER INSTRUCTIONS FOR SAFE OPERATION	4
1.6	INTENDED USE.....	4
2	PRODUCT DESCRIPTION	5
3	INSTALLATION AND SET UP	5
4	OPERATION	5
5	MAINTENANCE	5
6	CLEANING AND CONTAMINATION PROTECTION	6
7	TECHNICAL DATA	6
8	CE-CONFORMITY	6
9	MANUFACTURER:	6

1 General

To safely operate and control the ak 470 patient examination stool please carefully read this user manual before operating. Please pay attention to all cautionary statements pertaining to the safe operation of the ak 470. Please keep the manual on file at all times while the product is in service. The manufacturer reserves the right to make technical changes or modifications in accordance with advanced technical developments. This manual is not subject to an updating service.

1.1 Copyright

© Distributing, copying or any commercial use of this manual is strictly prohibited unless expressly permitted in writing by the manufacturer. The manufacturer is entitled to compensation for any violation of this right.

The manufacturer claims any and all rights in case a patent is granted or the design is registered.

1.2 Disclaimer

Manufacturer excludes any and all liabilities in case of improper or not authorized operation or maintenance or servicing of the product.

1.3 Applicable rules and regulations

The device complies with the following Regulations for machine products.

- 2006/42/EG
- RL 2011/65/ EU

The device meets the requirements of 2006/42/EG and the corresponding national requirements of the machinery directive.

1.4 Warning and cautioning signs

This sign indicates a hazard to the equipment!

This sign indicates a hazard to the patient, equipment, or user!

1.5 User instructions for safe operation

Before using this medical device, please carefully read and observe the safety instructions and recommendations listed in this manual

- Correct use and operation of the table is absolutely necessary for its safe function.
- Check power specifications on label and ensure compliance with local power supply.
- Connect ak 470 only to power sockets equipped with ground circuit. Any break of ground circuit (inside or outside of electrical equipment) may cause hazards to user or equipment. Never break ground circuit on purpose.
- Connection directly to the examination unit by a trained service technician only. In this case please observe a max. leakage current of 0,005 mA.
- Do not pull or tug on power cord.
- Do not operate the ak 470 series in environments exposed to hazardous materials. Do not operate in the presence of flammable anaesthetics and volatile solvents such as alcohol, benzene or similar agents.
- Do not set up the ak 470 series in humid or damp rooms. Avoid areas of sprinkling, dripping or splashing water.
- The ak 470 series should be operated by trained and authorized personnel only. Use other than designed use falls within the sole responsibility of operator.
- Assembly, service or modifications by authorized and trained staff only.
- The ak 470 series may be opened only by trained and authorized technicians. Disconnect power cord before opening equipment or changing fuses.
- When cleaning, make sure that no cleaning agent or water enters into the electric control box of the ak 470 series

1.6 Intended use

The medical device is intended for use as a patient chair. Any other use of the product is not permitted.

2 Product description

The Patient stool ak 470 is designed to serve as an electric height adjustable examination and treatment stool. The top swivels and locks mechanically every 90°.

The stool is controlled by 2 or 3 function foot switches or through a compatible examination unit.

Options:

Power: 110 / 120 Volts 50 / 60 Hz

Seat: round, low backrest, high backrest

3 Installation and Set up

Attention

If the stool is intended to be connected directly to the examination unit, the connection must be done only by a trained service technician.

- Place stool at its designated position. If necessary, level uneven floor.
- Check rotation area of top for any obstructions.
- Check automatic fuse unit for IN position.
- Connect foot switch to appropriate socket. (Plugs are configured to avoid mix up's).
- Connect power cord.

4 Operation

Stool height is adjustable through either foot switch or calipers on examination unit. The left section on foot switch controls UP, the right section DOWN movements. The optional center switch drives the AUTO Reset function. Just briefly press center switch for AUTO Down movement.

Pressing the left (UP) section will interrupt the AUTO Reset function.

Attention

The electric motor is protected by an integrated thermo protection switch. After 1 minutes of continued operation, the motor needs a cool down period of approx. 9 minutes. If the thermo protection switch activates, the motor needs a cool down period of approx. 20 minutes

5 Maintenance

The product is maintenance free for the end user **however**, bleed current and leak resistance need to be checked every other year or as directed by local laws.

6 Cleaning and contamination protection

- Contamination protection
In order to protect the product against contamination by liquids, body liquids or other unwanted substances it is recommended to, while in use, cover the product with a non sterile disposable impermeable cover sheet.
- Surfaces are resistant to all commonly used disinfectants as listed in the D G H M (Deutsche Gesellschaft fuer Hygiene und Mikrobiologie) list.
<http://www.gesundheitsamt.de/alle/technik/download/seuche/bakt/list/dghm.pdf>. Any appropriate disinfectant must not exceed the concentrations given for the following ingredients. Propanol=35% // Ethanol=25% This would equal for example Terralin Liquid offered by. Schülke & Mayr.
- Do not attempt to sterilize the ak 470
- Clean only the external surfaces of the equipment using a damp cloth. In case of heavy stains do **not** use abrasive or aggressive materials other than common cleansers or detergents. Do not use ether, acetone or concentrated acids.
- No water or cleaning detergents may enter the inside of the equipment.
- To clean soiled upholstery surfaces, use a damp cloth and/or commonly used upholstery spray.
- Before resuming operation, allow sufficient time to dry.

7 Technical Data

Electric motor:	220V / 50Hz (120V / 60Hz)
Thermo protection switch	3 A (220V)
Thermo protection switch	6 A (120V)
Rated current	5,5 A
Vertical lift	200 mm
Vertical speed	12 mm/Sek
Lifting capacity	150 Kg
Min. seat height	470 mm
tll. dimensions (approx)	(W x H x D):..... 650 x 1360 x 650 mm
KB	1 min/10 min

Product is subject to changes

8 CE-conformity

We declare the compliance of the device with the requirements of the Council Directive 2006/42/EG machinery directive

9 Manufacturer:

AKRUS GmbH & Co. KG

Otto-Hahn-Str. 3

25337 ELMISHORN

☎ int. +49 (0) 4121 791930

FAX int. +49 (0) 4121 791939

Email: info@akrus.de

www.akrus.de