Instrumententisch Instrument Table

ak 103 a / s / v

Gebrauchsanweisung User Manual

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User Manual

Table of contents

1.	Applicable standards and regulations.....	3
2.	Installations instructions and use.....	3
3.	Product information labels and warning stickers.....	4
4.	Intended use	4
5.	General, construction, controls.....	4
6.	Set up and leveling the instrument table.....	5
7.	Main power connection.....	5
8.	Adjusting the table height.....	5
9.	Maintenance	5
10.	Changing of fuses.....	6
11.	Cleaning and Care	6
12.	Specification.....	7
13.	Manufacturer:	8
14.	Declaration of conformity.....	8

1. Applicable standards and regulations

- The ak 103 series have been designed in compliance with and to meet or exceed the requirements of
 - EN 60601-1 and UL 2601
 - Class I products in accordance with the German Medical Products Law
 - Radio interference certification EN 55011/Class B
 - Electromagnetic compatibility EN 60601-1-2
 - Directive 93/42/EWG annex IX
 - 
- Observe the applicable national regulations for the prevention of accidents.

2. Installations instructions and use

- **Before using the instrument table, please carefully read and observe the safety instructions and recommendations listed in this manual**
- Correct use and operation of the table is absolutely necessary for its safe function.
- Check power specifications on label and ensure compliance with local power supply.
- Connect table only to power sockets equipped with ground circuit. Any break of ground circuit (inside or outside of electrical equipment) may cause hazards to user or equipment. Never break ground circuit on purpose.
- Do not pull or tug on power cord.
- Do not operate the ak 103 series in environments exposed to hazardous materials. Do not operate in the presence of flammable anaesthetics and volatile solvents such as alcohol, benzine or similar agents.
- Do not set up the ak 103 series in humid or damp rooms. Avoid areas of sprinkling, dripping or splashing water.
- The ak 103 series should be operated by trained and authorized personnel only. Use other than designed use falls within the sole responsibility of operator.
- Assembly, service or modifications by authorized and trained staff only.
- The ak 103 series may be opened only by trained and authorized technicians. Disconnect power cord before opening equipment or changing fuses.
- When cleaning, make sure that no cleaning agent or water enters into the electric control box of the ak 103 series

3. Product information labels and warning stickers

Position:

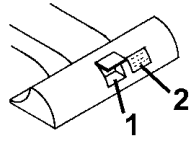
Fig. 1

1. Power plug
2. Label

This sign means:
Hazards to the instrument

This sign means:
Hazards to the user

This sign means:
Instrument type B according to IEC 601-1

4. Intended use

The ak 103 series instrument tables are vertically adjustable. They have been designed to carry ophthalmic instruments up to a max. permissible table load of 35 kg.


The power connection unit is located under the table top: 230V Volt, max. current 2,5 A! Ensure proper footing and balance of table considering weight of instruments or equipment (IEC 601-1 / 24). The combination of both, the instrument table and the optical equipment, must not exceed max ground current of 0,5 mA.

5. General, construction, controls

Infinitely variable height adjustments allow to precisely position tabletop at its required height for convenient, fatigue-free operation. Height adjustment by an enclosed maintenance-free electric motor drive.

Switch for adjusting the table height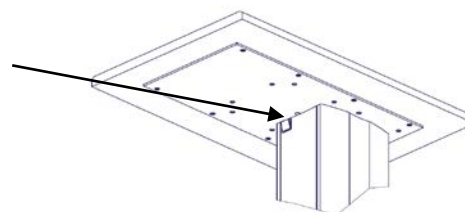

6. Set up and leveling the instrument table

- Assure solid and firm footing, avoid any danger of tipping. Level the tabletop by turning the vertically adjustable feet (1) or castors.
- Secure the position after leveling using the lock nuts (2).

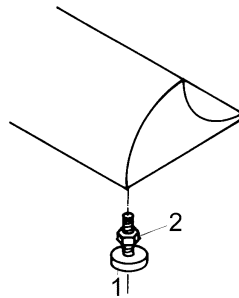
Fig. 3

7. Main power connection

Make sure specified power limitations on the type label comply with the local power supply. If not, do not connect the instrument table.

8. Adjusting the table height

An electric motor drive inside the center column allows vertical adjustment of the tabletop. Position tabletop at desired height.

Attention – clamp danger !

Caution: Overheat protection!

The motor in the lifting column is not designed for continuous long-term operation. Therefore, lift or lower the table only, when necessary. Operating periods of > 1 minute may cause overheating of the motor.

9. Maintenance

The ak 103a, ak 103s, ak103v is maintenance free.

10. Changing of fuses

The fuses are located inside the power connecting box..


Attention: Prior to commencing any work on the electric or electronic parts of the product, **disconnect the power cord.!**

Caution: use only fuse in compliance with the technical data specified on technical data label

- Replace defective fuses as shown in Fig. 4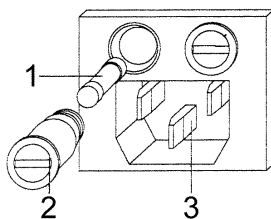

- 1 – fuse
- 2 – fuse holder
- 3 – cable plug

Should fuse blow again after switching on the unit, a major electric problem has occurred. Please contact a qualified service technician.

11. Cleaning and Care

Clean only the external surfaces of the instrument table using a damp but not wet cloth, in case of heavy stains do not use abrasive or aggressive materials but regular cleaning detergents for cleaning. Do not use ether, Acetone or concentrated acids.

Caution: do not use any solvents (ether, acetone), acids or abrasive cleaning agents.

12. Specification

Floor space:	420 x 650mm (ak103v – 520 x 1000)
Table top:	420 x 500mm (ak103v – 420 x 1000)
Min. height:	640mm (on castors + 35mm)
Max. height:	940mm (on castors + 35mm)
Height of lift:	300mm
Lifting speed:	13 mm/s
Weight:	~ 31 kg
Max. permissible table load:	50 kg
Max. motor load:	~ 1500 N
Hz frequency:	50Hz
Class of protection and Rating:	SK I / IP 21
Fuses electric power 230V:	T 10,0 A, / 250V
Ophthalmic Instruments Power supply on 120V:	2 x 2,5 A
Environmental conditions	
For intended use:	Temperature +10...+40°C; Humidity 30...75%; Air pressure 700...1060 hPa
Storage conditions	
(original packing):	Temperature -10...+55°C; Humidity 10...95%; Air pressure 700...1060 hPa
Shipping conditions	
(original packing):	Temperature -40...+70°C; Humidity 10...95%; Air pressure 500...1060 hPa

13. Manufacturer:

AKRUS GmbH & Co. KG

Otto-Hahn-Str. 3

25337 ELSHORN

☎ int. +49 (0) 4121 791930

FAX int. +49 (0) 4121 791939

Email: info@akrus.de

www.akrus.de

14. Declaration of conformity

We declare the compliance of the device with the requirements of the Council Directive 93/42/EEC about Medical Devices

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1.1. Copyright