Instrumententische Instrument Table

ak 106 vario

Bedienungsanleitung
User Manual

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User Manual

Table of contents

1	GENERAL	3
1.1	COPYRIGHT	3
1.2	DISCLAIMER.....	3
1.3	APPLICABLE RULES AND REGULATIONS.....	3
1.4	WARNING AND CAUTIONING SIGNS	3
1.5	USER INSTRUCTIONS FOR SAFE OPERATION.....	4
2	INTENDED USE	4
3	PRODUCT INFORMATION LABELS AND POWER SUPPLY	5
4	GENERAL, CONSTRUCTION, CONTROLS	5
5	SET UP AND LEVELLING THE INSTRUMENT TABLE	6
6	MAIN POWER CONNECTION	6
7	ADJUSTING THE TABLE HEIGHT	6
8	MAINTENANCE	6
9	CLEANING AND CARE	6
10	TECHNICAL DATA.....	7
11	DECLARATION OF CONFORMITY	7
12	MANUFACTURER.....	7

1 General

To safely operate and control the ak 106 vario please carefully read this user manual before operating. Please pay attention to all cautionary statements pertaining to the safe operation of the ak 106 vario. Please keep the manual on file at all times while the product is in service.

The manufacturer reserves the right to make technical changes or modifications in accordance with advanced technical developments. This manual is not subject to an updating service.

1.1 Copyright

© Distributing, copying or any commercial use of this manual is strictly prohibited unless expressly permitted in writing by the manufacturer. The manufacturer is entitled to compensation for any violation of this right.

The manufacturer claims any and all rights in case a patent is granted or the design is registered.

1.2 Disclaimer

Manufacturer excludes any and all liabilities in case of improper or not authorized operation or maintenance or servicing of the product.

1.3 Applicable rules and regulations

The device complies with the following Regulations for machine products.

- 2006/42/EG
- RL 2011/65/ EU

The device meets the requirements of 2006/42/EG and the corresponding national requirements of the machinery directive.

User must observe the national legal requirements governing the prevention of accidents.

1.4 Warning and cautioning signs

This sign indicates a hazard to the equipment!

This sign indicates a hazard to the patient, equipment, or user!

1.5 User instructions for safe operation


Before using this medical device, please carefully read and observe the safety instructions and recommendations listed in this manual

- To safely use and operate the medical device, use only as directed.
- Operation of the medical device only in accordance with the intended use.
- Do not operate the medical device in explosive or hazardous environments. The medical device must not be operated, with the exception of very minute quantities, in the presence of combustible anesthetics, or volatile solvents like alcohol, benzene or the like.
- Do not set up the medical device in humid or damp rooms. Never expose the product to sprinkling, dripping or splashing water.
- Modifications, authorized by manufacturer, may only be performed by staff authorized by manufacturer. Any modification non authorized or instructed by manufacturer may cause malfunctions or hazards to persons.
- Maintenance or repair work on this product may only be performed by staff authorized by manufacturer.
- Pay special attention to any emphasized safety instruction or information in this user manual.
- Only trained and authorized staff may operate the medical device.
- Only trained service staff authorized by manufacturer may open the medical device.
- No water or other liquids may enter the electric connector housing during maintenance or cleaning procedures. Please refer to section "Cleaning and care".

2 Intended use

The ak 106 vario instrument tables are vertically adjustable. They have been designed to carry ophthalmic instruments up to a max. permissible table load of 50 kg.

3 Product information labels and power supply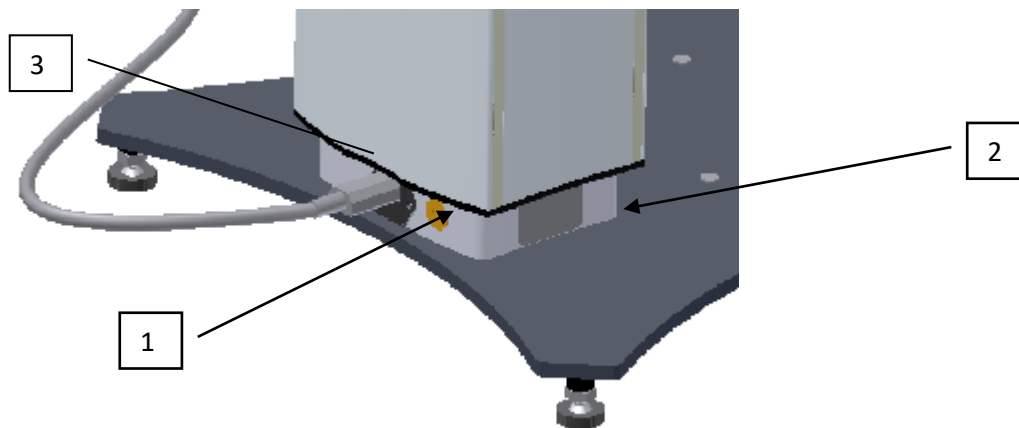

Position:

- 1) power plug
- 2) label
- 3) voltage switch 120V-230V (option)


This sign means:
Hazards on the instrument

4 General, construction, controls


Infinitely variable height adjustments allows to precisely position tabletop at its required height for convenient, fatigue-free operation. Height adjustment by an enclosed, maintenance-free electric motor.

1. Switch for adjusting the table height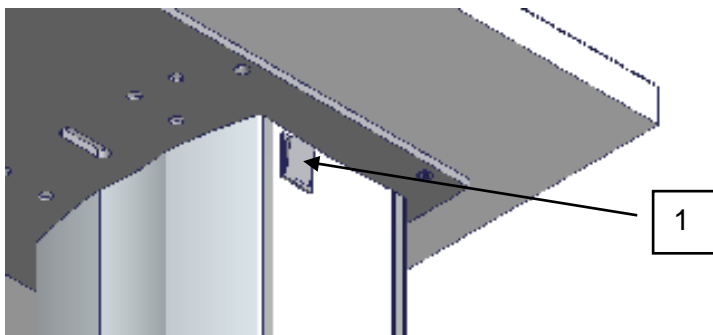

5 Set up and levelling the instrument table

- Assure solid and firm footing, avoid any danger of tipping. Level the tabletop by turning the vertically adjustable feet (1) or castors.
- Secure the position after leveling using the lock nuts (2).

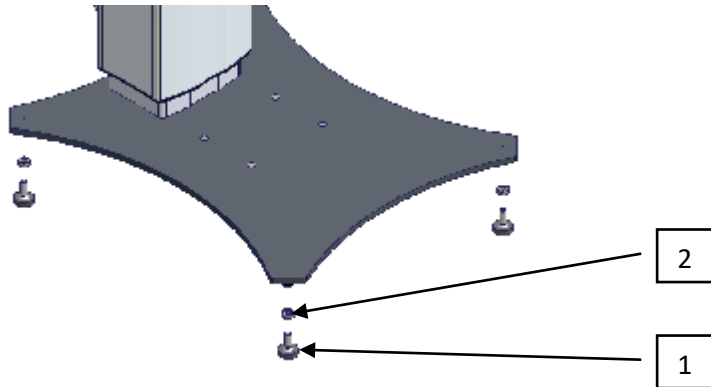6 Main power connection

Make sure specified power limitations on the type label comply with the local power supply. If not, correct the setting of the voltage switch (option, see chapter 3).

Caution! The setting of the voltage switch is important because a wrong setting can cause seriously damage of the motor column. This damage could only be repaired by the manufacturer.

7 Adjusting the table height

An electric motor drive inside the center column allows vertical adjustment of the tabletop. Position tabletop at desired height.

Caution: Overheat protection!

The motor in the lifting column is not designed for continuous long-term operation. Therefore, lift or lower the table only, when necessary. Operating periods of > 1 minute may cause overheating of the motor.

8 Maintenance

The ak 106 vario is maintenance free.

9 Cleaning and care

Clean only the external surfaces of the instrument table using a damp but not wet cloth, in case of heavy stains do not use abrasive or aggressive materials but regular cleaning detergents for cleaning. Do not use ether, Acetone or concentrated acids.

Caution: do not use any solvents (ether, acetone), acids or abrasive cleaning agents.

10 Technical data

Floor space:	420 x 520mm
Table top:	500 x 600mm (V=560mm X 1000mm; large 600 x 1000)
Height:	min 615 mm – max 915mm (on castors)
Height of lift:	300mm
Lifting speed:	13 mm/s
Weight:	~ 39 kg
Max. permissible table load:	50 kg (central over the motor)
Max. motor load:	~ 1500 N
Power connection	230V ± 10% / 120V ± 10%
Hz frequency:	50/60Hz
Max. current	max. 1.5 A
Rated power	120 W
Class of protection and Rating:	SK I / IP 21
Environmental conditions	Temperature +10...+40°C; Humidity 30...75%; Air pressure
For intended use:	700...1060 hPa
Storage conditions	Temperature -10...+55°C; Humidity 10...95%; Air pressure
(original packing):	700...1060 hPa
Shipping conditions	Temperature -40...+70°C; Humidity 10...95%; Air pressure
(original packing):	500...1060 hPa

11 Declaration of conformity

We declare the compliance of the device with the requirements of the Council Directive 2006/42/EG machinery directive

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Position of lift

Small top, 500 x 600, center or off center

Large top 600 x 1000 (rectangular or V shape) ONLY CENTER

2. Place table top (Pos. 4) upside down on the floor, place table top mounting plate on the electric lift on table top, line up bores and tighten the 8 Spax- screws (Pos. 100).
3. Screw studs into base and secure with the lock nuts (Pos. 15, 80)..
Do not use lock nuts on the optional castors (Pos. 16).
4. Attach base (Pos. 1) to electric lift (Pos.10) using the 4 cylinder head screws M8x30 (Pos. 90, 60) .
Caution Make sure the base is lined up with the table top mounting plate. (center or off center)
5. Insert plastic covers (Pos. 17) into the spare holes, secure loose cables with wire clamps (Pos.12) under the table top.

Pos. Nr.	Artikel Nr.	Benennung
1	444.010 003	Base plate
2	444.031 005	Table top adapter
4	442.050 001 442.050 002 442.050 003	Table top 500x600x28 Table top V- Form Table top 600x1000
10		Electric lift
11	100-020	Main cable
15	320-115	Adjustable stands
16	320-114 + 320-116	Bolt Roller
17	330-289	Plastic cover
60	410-025	Washer 8,4
70	793-080	Countersunk screw M8x35
80	475-030	Hex head nut M10
90	445-100	Socket head screw M8x30
100	499-092	Spax screws 4,5x25

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akrus GmbH & Co KG